

Remote Microphone Connection Procedures for the Cochlear Kanso 2 – Cochlear Mini Mic 2+ as Transmitter

1. **PAIR** the MM2+ to the Kanso 2. (**PAIRING** only needs to be done at the initial set up; **PAIRING** holds until MM2+ is paired to another processor or another accessory is paired to Channel 1 on Kanso 2)
 - a. Turn OFF the Kanso 2 by tapping the Cochlear logo 3 times. The indicator light turns ORANGE.

Triple-tap (3 taps – quick and firm).



- b. Turn ON the MM2+ by pressing the ON/OFF button for 2 seconds until the LED turns GREEN, then release the button.



- c. With the tip of a ballpoint pen, press the pairing button on the back of the MM2+ once. The LED will flash YELLOW. Within 20 seconds, turn on the Kanso 2 by tapping the Cochlear logo twice. The light on the Kanso 2 turns GREEN.



Double-tap (2 taps – quick and firm).



Daily Classroom Routine

Before instruction begins:


1. Teacher to position the MM2+ about 20 cm below his/her lips using the supplied lanyard.
2. Turn ON the MM2+.

3. Activate **STREAMING** on the Kanso 2 using either the *CR310 Remote Control* or the *Nucleus Smart* app on an *Apple* or *Android* device.

CR310: 2 second hold on T button

OR

Nucleus Smart: tap “Mini Mic” under “Audio Sources”

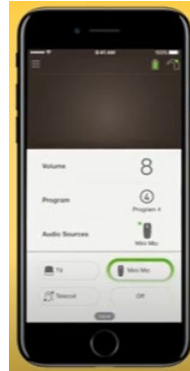

The Kanso 2 blinks BLUE when **STREAMING** is activated.

- 6) Remote microphone connection is now established. Conduct Functional Check.

Functional check

From a distance of at least 3 metres, cover your lips and speak to the student via the Roger Touchscreen. Ask the student to do things like “point to the windows” or “touch your nose” and check for understanding.

During Instruction Breaks (e.g. quiet study, recess, lunch):

1. Teacher to mute the MM2+ by pressing the Mute button once. An orange light will blink around the MUTE button.
2. When instruction resumes, unmute the MM2+ by pressing the Mute button once.

At the end of the school day:


1. Turn OFF the MM2+ by pressing and holding the POWER button for 2 sec until a red light comes on, then release the button.



2. Charge both devices in a safe, dry, and clean place.

3. Student to toggle OUT of **STREAMING** using either the *CR310 Remote Control* or *Nucleus Smart* app.

CR310: quick press on T button OR *Nucleus Smart*: tap OFF under “Audio Sources”

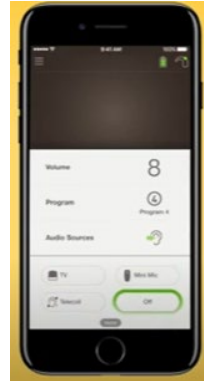

Auditory Outreach