

Remote Microphone Connection Procedures for the Cochlear Kanso 2 – Phonak Roger Touchscreen as Transmitter & Cochlear Mini Mic 2+ as Intermediary

1. **PAIR** the MM2+ to the Kanso 2. (**PAIRING** only needs to be done at the initial set up; **PAIRING** holds until MM2+ is paired to another processor or another accessory is paired to Channel 1 on Kanso 2.)
 - a. Turn OFF the Kanso 2 by tapping the Cochlear logo 3 times. The indicator light turns ORANGE.

Triple-tap (3 taps – quick and firm).



- b. Turn ON the MM2+ by pressing the ON/OFF button for 2 seconds until the LED turns GREEN, then release the button.



- c. With the tip of a ballpoint pen, press the pairing button on the back of the MM2+ once. The LED will flash YELLOW. Within 20 seconds, turn on the Kanso 2 by tapping the Cochlear logo twice. The light on the Kanso 2 turns GREEN.



Double-tap (2 taps – quick and firm).



2. **CONNECT** the Roger X receiver to the Roger Touchscreen transmitter. (**CONNECT** needs to be done at the initial set up only; connection holds until the Roger X is connected to another Roger transmitter.)
 - a. Turn ON the Roger Touchscreen by pressing and holding the side power button for 3 sec.



- b. Insert the Roger X into the FM connector of the powered-on MM2+.



- c. While holding the powered-up Roger X within 1 inch of the Roger Touchscreen, tap CONNECT.



- d. If the operation is successful, the message "Roger X is connected" will appear on the Roger Touchscreen.


Daily Classroom Routine

Before instruction begins:

1. Teacher to position the Roger Touchscreen about 20 cm below his/her lips using the supplied lanyard.



2. Turn ON the Roger Touchscreen.
3. Turn ON the MM2+ with the Roger X inserted in the FM connector. For optimum RM reception, the MM2+ should stay within 8 metres and in the same room as the Kanso 2 wearer. The reception range between Touchscreen and Roger X is up to 20 metres indoors as long as the speaker and student are in the same room with no physical barriers, and up to 40 metres outdoors.
4. Press the Mode button on the MM2+ until the FM icon is lighted green.

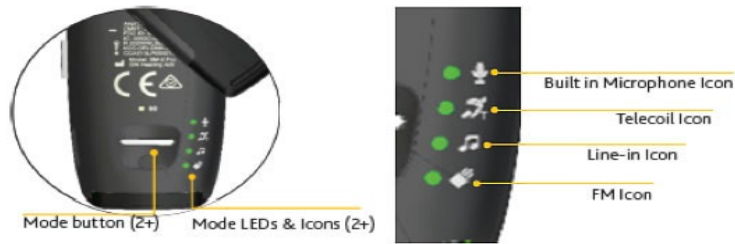


5. Activate **STREAMING** on the KANSO 2 using either the *CR310 Remote Control* or the *Nucleus Smart* app on an *Apple* or *Android* device.

CR310: 2 second hold on T button

OR

Nucleus Smart: tap “Mini Mic” under “Audio Sources”

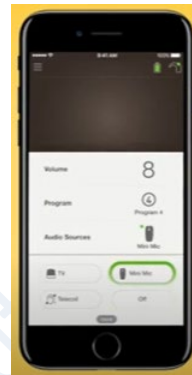

The KANSO 2 blinks BLUE when **STREAMING** is activated.

6) Remote microphone connection is now established. Conduct Functional Check.

Functional check

From a distance of at least 3 metres, cover your mouth and speak to the student via the Roger Touchscreen. Ask the student to do things like “point to the windows” or “touch your nose” and check for understanding.

During Instruction Breaks (e.g., quiet study, recess, lunch):

1. Teacher to mute the Roger Touchscreen by pressing the Mute button once. A RED light will blink around the MUTE button.



2. When instruction resumes, unmute the Roger Inspiro by pressing the Mute button once.

During small group work (up to 6 people around a table):

1. Activate the automatic conference mics on the Touchscreen by placing it on a flat surface at the centre of the group.



At the end of the school day:

1. Turn OFF the Roger Touchscreen by pressing and holding the power button for 3 sec and tap on the upper right check mark for Yes.



2. Turn OFF the MM2+ by pressing and holding the POWER button for 2 sec until a red light comes on, then release the button.



3. Charge both devices in a safe, dry, and clean place.
4. Student to toggle OUT of **STREAMING** using either the *CR310 Remote Control* or *Nucleus Smart* app.

CR310: quick press on T button OR *Nucleus Smart*: tap OFF under “Audio Sources”

