

Roger NeckLoop Connection to Smartphone with Lightning Socket

For Speech to Text Purposes



You need: The included USB-C to USB-A cable (shown in black above), a Lightning to USB 3 Camera Adapter (not supplied), and your phone charger.

1. Connect the USB 3 Camera Adapter to your smartphone.
2. Connect your phone charger to the adapter.
3. Connect Roger NeckLoop to the adapter with the included USB-C to USB-A cable.
4. Turn Roger NeckLoop on, connect to Roger Transmitter (Roger Touchscreen, Roger ON)
5. Run the speech-to-text app on your smartphone or tablet (ie. Scribeberry, Word, Google Docs, etc.)
6. If necessary, select Roger NeckLoop as input microphone source on your phone.
7. Speak into the Roger microphone and the speech-to-text app will transcribe the spoken words into text.