



Auditory Outreach... A Provincial Resource Program

4351 Ontario Avenue, Powell River, BC V8A 1V3

Toll Free: 1-866-430-4327

Fax: 604.485.2886

www.auditoryoutreach.ca

Student: _____

Date of Birth: _____

School: _____

Teacher: _____

Phonak DigiMaster 5000 Speaker



Status Indicator

Constant blue light	DigiMaster is on and receiving signal from Inspiro
Blinking blue light	Stand-by mode
Blinking yellow light	Inspiro not paired
Constant green light	Inspiro mic muted
Constant red light	DigiMaster needs to restart

Auditory Outreach

On/Off Switch and Audio Input
(bottom of DigiMaster)

Trouble Shooting

Issue	Things to Try
No Sound	<ol style="list-style-type: none"> 1. Check the light indicators. 2. Ensure that the system is plugged in to a power source. 3. Ensure all cords have a sturdy connection. 4. Restart the DigiMaster (turn it on/off).
Intermittent Signal	<ol style="list-style-type: none"> 1. Ensure all cords have a sturdy connection. 2. Check the Inspiro EasyBoom microphone placement and function; replacement needed?
Static	<ol style="list-style-type: none"> 1. Ensure all cords have a sturdy connection. 2. Check the Inspiro EasyBoom microphone placement and function; is a microphone replacement needed?

...Please contact Auditory Outreach if the above steps do not resolve the issue.