

SET UP GUIDE FOR EASYLINK S AND MLxS

GETTING STARTED-Charging The System

1. Plug charging unit into the wall
2. Attach EasyLink Adaptor
3. Plug the charging Unit into the power jack on the adaptor marked "DC 7.5V"
4. Charge for minimum 1 hour



SWITCHING EASYLINK S ON/OFF

1. Press and hold on/off button for 2 seconds.



CHANNEL CHECK

1. Confirm that channel on EasyLink matches channel of MLxS receiver
2. If channels do not match, insert a pen into hole on back of EasyLink. Hold down until desired channel appears.

Insert pen here to change channel



ATTACH RECEIVER TO AUDIO SHOE

1. Attach MLxS receiver to audio shoe
2. Attach Audio shoe to hearing aid
3. Switch to two dots (FM + M)
4. Switch Hearing Aid On

TROUBLESHOOTING: NO FM?

1. Try a new battery
2. Re-attach audioshoe with hearing instrument
3. Re-attach the MLxS and audioshoe
4. Verify that all switches are set correctly
5. Verify that EasyLink S is on the correct channel

HEARING AID NOT WORKING?

1. Is tubing cracked or earmold block with moisture/debris?
2. Replace the battery
3. Clean earmold

INTERMITTENT OR STATIC?

1. Is lavalier cord/ antenna being used at full length (not wound up)?
2. Wiggle audio shoe and receiver. Is it loose? Replace audio shoe
3. Listen to FM and HI as you move around the room. Is there something in the room generating interference (e.g. flickering fluorescent lighting)?
4. Change the transmitter channel and synchronize to the receiver to a new channel. Check for static on this channel.