

## EASY GAIN SAVING IN INSTALLED ROGER X RECEIVERS

### Did you know?

For hearing aids with *Roger Direct* (i.e., those that take installed *Roger Xs*), changes made to the receiver gain, or *Easy gain*, are saved in the **hearing aid** and not the receiver.


### What this means ...

Once an installed *Roger X* is verified in a *Roger Direct*-compatible hearing aid, the final adjusted gain stays with the hearing aid.



### SCENARIO ONE:

Student A has a Sky Marvel hearing aid in which there is suspected RM malfunction. The Roger X is uninstalled and sent to AO-PRP for service. The Roger X is replaced by another Roger X with a different serial number.

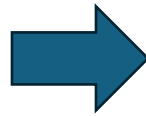

---

Re-verification or gain adjustment is not required. Once re-installed, the replacement Roger X will automatically set to the final gain adjusted at last verification.

---

### SCENARIO TWO:

Student B has been using Roger with a non-*Roger Direct* hearing aid and has now received a new hearing aid that has *Roger Direct*.



---

Despite staying with the same Roger system, student B needs to see the audiologist for RM verification.

---

### SCENARIO THREE:

Student C has binaural hearing aids with *Roger Direct* and installed Roger X receivers. At verification, it is determined the left receiver needs a -4dB offset while the right receiver meets transparency at 0dB.



---

For record-keeping, the receiver gain should be recorded to the hearing aid and not the receiver. When the receivers are uninstalled, e.g., for vacation or service, it is no longer important to mark the receivers as L or R. Once re-installed, a Roger X receiver automatically sets to the last gain saved in the hearing aid.

---