


## **Description MLxS**

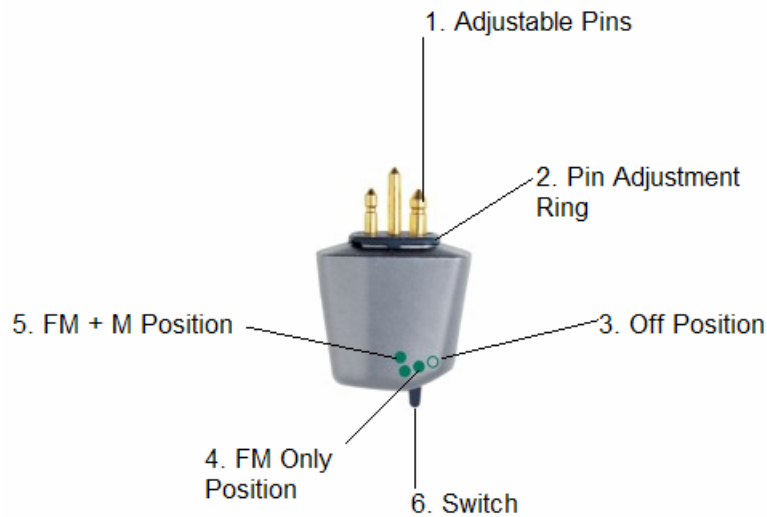

1. Adjustable Pins. Pins can be adjusted for crosswise or lengthwise configuration depending on how the audio shoe of the hearing aid is oriented.
2. Pin Adjustment Ring. To change the pin orientation, pull this ring up gently and turn the pins to the new position.
3. Off Position. Turns receiver off. Also returns receiver to default wake-up frequency when turned back on again.
4. FM Only Position. Allows student to hear teacher only, but not own voice or voice of classmates.
5. FM + M Position. Recommended setting for most students. Allows student to hear teacher as well as own voice and classmates.
6. User Switch.

## **Stand-by Mode**

The stand-by mode is an energy saving feature of the MLxS. When the transmitter is switched off or goes out of range, the MLxS receiver automatically enters stand-by mode 30-40 seconds later. Virtually no power is consumed in this mode. The receiver detects when the transmitter is switched on again and automatically resumes normal operation. It may take up to 4 seconds for the MLxS to switch on again. Note that the receiver remembers the last frequency used; therefore, there is no need to re-synchronize the receiver to the transmitter frequency.