

Remote Microphone Connection Procedures for the Nucleus 7 with Cochlear Mini Mic 2+ as Transmitter

1. Pair the Nucleus 7 to the MM2+ (pairing only needs to be done once unless another accessory was paired to Channel 1 of the N7).
 - a. Turn off the Nucleus 7 by pressing and holding the button for 5 seconds until the indicator light turns a steady orange, then release the button.



- b. Turn on the MM2+ by pressing the ON/OFF button until the LED turns green, then release the button.



- c. With the tip of a ballpoint pen, press the pairing button once. The LED will flash yellow. Within 20 seconds, turn on the N7 by pressing the button on the N7 quickly once. A green light will appear, followed by a blue light, which indicates successful pairing.



2. Activate **STREAMING** on the N7 by pressing the N7 button for 2 seconds and release. The N7 light will blink blue.

3. Functional Check

- a. Have the student put the N7 back on the implant.
- b. From a distance of at least 3 metres, cover your mouth and speak to the student via the MM2+. Ask the student to follow instructions light “point to the windows” or “touch your nose” and check for understanding.


Daily Classroom Routine:

Before instruction begins:

1. Teacher to position the MM2+ about 20 cm below their mouth by using either the neck loop or clip.
2. Turn on the MM2+.
3. Student to activate streaming on the N7.
4. Teacher to conduct functional check.

During Short Instruction Breaks (e.g. quiet study, individual work time):

1. Teacher to mute the MM2+ by pressing the Mute button once.
2. LED on MM2+ will flash orange.

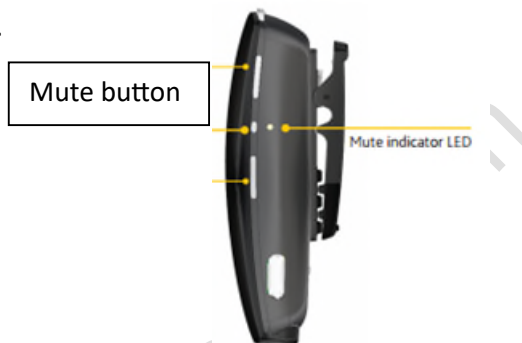

3. When instruction resumes, unmute the MM2+ by pressing the Mute button once. The LED will stop blinking.

During Longer Breaks (e.g. recess, lunch):

1. Teacher to turn off the MM2+ by pressing the ON/OFF button until the LED turns red, then release the button.
2. Student to toggle N7 out of streaming by pressing the N7 button once quickly. The LED will blink green once.



During small group work (up to 5-6 people around a table):

1. Place the MM2+ on a flat surface at the centre of the group.
2. The omni-directional mics on the MM2+ are automatically activated when placed in a horizontal position.

At the end of the school day:

1. Turn off the MM2+ and charge it in a safe, dry place.
2. Student to toggle out of streaming.