

How to generate a receiver in a hearing aid using the Roger Touchscreen v3:


- 1) Ensure hearing aid has Roger Direct technology. The following models by Phonak are compatible: Marvel, Paradise, Lumity, Infinio. The Unitron Blu and DiscoverNext are also compatible.

- 2) Power ON the Touchscreen v3 by pressing and holding the side button for 3 seconds.



- 3) Swipe screen LEFT.



- 4) Tap Roger Direct.

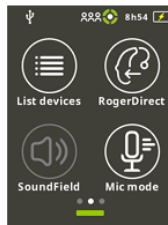

- 5) Bring powered-on hearing aid within **2 inches** of the Touchscreen v3 and tap Play.

- 6) A check mark indicates a receiver is successfully generated.

- 7) Tap  to return to home screen.

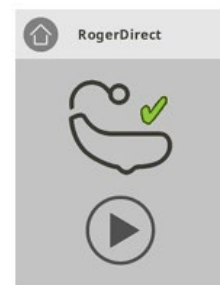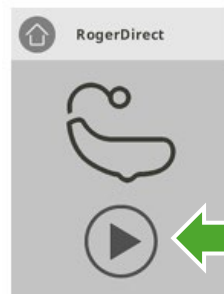

Important note: any hearing aids or Phonak transmitters/receivers should be turned OFF or at least 3 metres away from the hearing aid you are generating receiver on.

- 8) *Connect* receiver to start RM streaming. Bring Touchscreen to within 3 inches of the powered-on hearing aid and tap “Connect” on the Touchscreen.

- 9) A message “Roger Rx is connected” indicates Touchscreen is transmitting to the newly genated receiver.

