

Remote Microphone Connection Procedures for the Med El Rondo 3 Audio Processor

– AudioLink as Transmitter



Initial Set up:

1. Fully charge the AudioLink. Open the socket cover at the bottom of the AudioLink. Plug the charging cable into the micro USB socket. Light is orange when charging and green when fully charged.



2. Pair the AudioLink to the Rondo.

- a. Power ON the AudioLink by pressing and holding the  button for 2 seconds.
- b. Power OFF the Rondo. Put the magnet side of the Rondo on the lower part of the AudioLink. Power ON the Rondo. If successfully paired, the bracket symbols will light up.




Daily procedures:

- 1) Power ON the AudioLink.
- 2) Press the Mic button. The bracket and microphone symbols will light up to indicate successful streaming.



- 3) During instruction breaks, press the Mic button again to stop streaming.

- For best reception of the teacher's voice, AudioLink should be worn using the lanyard by the teacher or clipped onto their clothing so it is within 1 metre of their mouth.

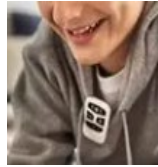

- Transmission range is up to 3 metres between AudioLink and Rondo. Student should be seated within 3 metres of the teacher to benefit from its remote mic function.
- In small group work, the AudioLink can be placed in its docking station and positioned in the middle of the group to improve the Rondo wearer's hearing of those seated around the table.



At the end of the school day:

- a) Power OFF the AudioLink by pressing and holding the  button for 2 seconds. The light will go off.
- b) Charge the AudioLink using the included USB cable and charger. Light is orange when charging and turns green when fully charged (about 3 hours).